

SOFTLINE COLLAR & BRIDGE KIT


32685

INSTALLATION & USAGE

1. Place **Collar Assembly** around neck; fasten buckle or press hook-n-loop fabrics together (A).



2. If distance of pre-assembled **Bridge Assembly** (B) to face isn't appropriate for usage, determine which set of five pre-drilled holes (C) will best suit user. Remove Collar. Loosen and remove screw and washer combination (D) holding Bridge Assembly. Reposition Assembly over appropriate set of holes. BEFORE REPLACING ASSEMBLY, DETERMINE WHETHER SPACERS ARE NEEDED TO ACCOMMODATE RESPIRATOR TUBES, WIRES, OR CORDS; IF SO, SEE STEP 3 BELOW. Re-secure to collar by replacing screw and washer combination.

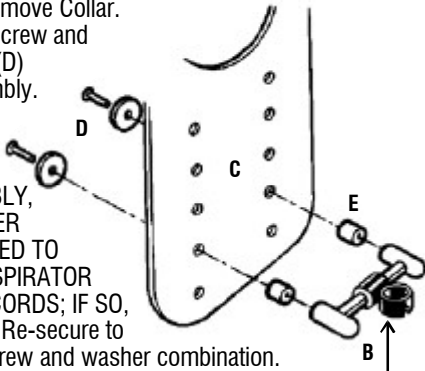


Adaptor Clamp shown here is not included with either Collar and Bridge.


3. If you need to adjust or increase Bridge Assembly height, use **Plastic Spacers** (E) provided. Unscrew Bridge Assembly and insert Spacers as needed. Reassemble using longer bolts provided.

CUSTOM FITTING

4. To custom-fit for comfort, place masking tape over areas to be trimmed. Indicate with pencil or marker areas to be cut away. Trim to line using scissors. Sand rough edges smooth with fine grade sandpaper.

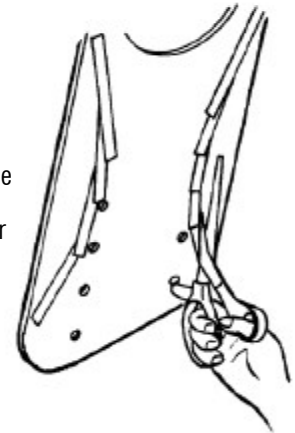


5. If desired, custom fit pre-formed collar by heating with "hot air gun" (a hand-held blow dryer is generally sufficient). Heat area to be re-shaped using high heat setting; CAUTION: PLASTIC CAN BECOME VERY HOT. RECOMMEND USE OF PROTECTIVE PADDING ON HANDS WHEN RE-FORMING USING HEAT. When sufficiently warm, gently mold plastic into desired shape; hold briefly until plastic cools and sets.

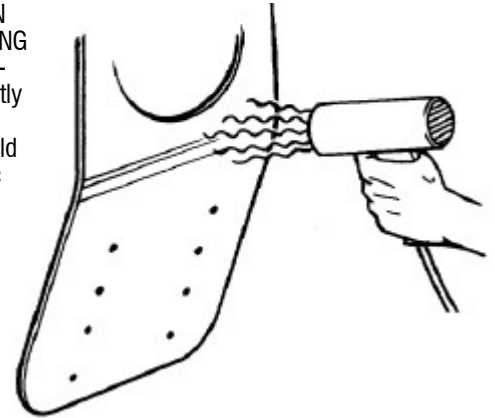

TO USE AN ADAPTOR CLAMP (NOT INCLUDED) WITH COLLAR & BRIDGE ASSEMBLY

Unscrew one side of **Bridge Assembly** from collar. Slide white plastic **Riser** (F) off bar. Slide an adaptor clamp (G, not included) onto bar. Replace Riser, and re-secure using screw and washer combination. Once final position is determined, tighten adaptor clamp in place.

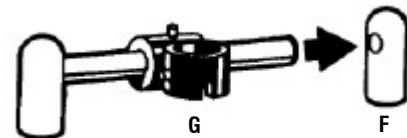

9450 W Laraway Rd • Frankfort, IL 60423
Customer 800-843-7234 • Fax 888-479-1515
www.TheraFin.com • info@TheraFin.com