


# Seat Cushion Communication Mount

- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.


## Product Identification

32374 Seat Cushion Communication Mount

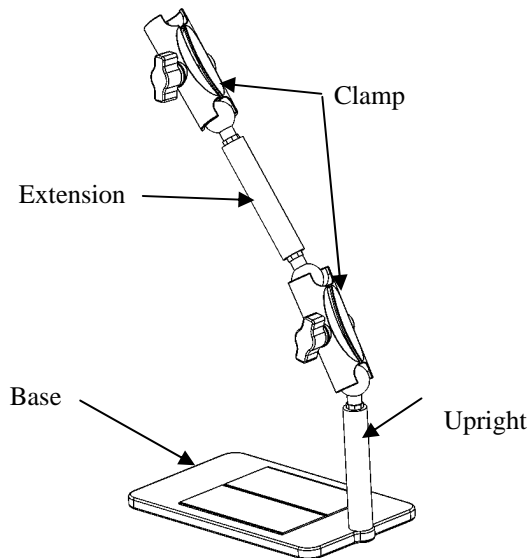


## To Install:


Image below shows installed device.

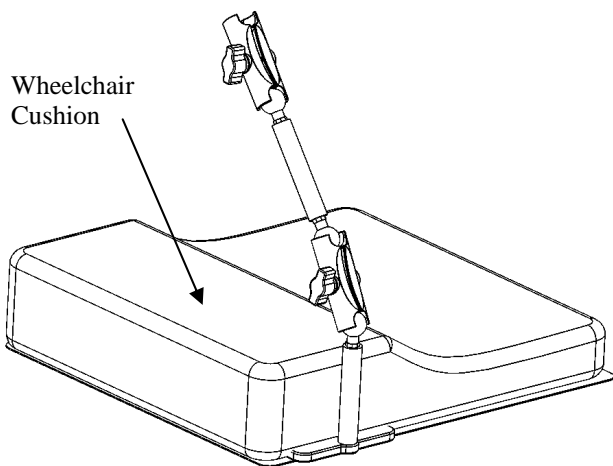

**Step 1:** Place the Base in the desired position between the Cushion and the wheelchair Sling or Cushion Platform.

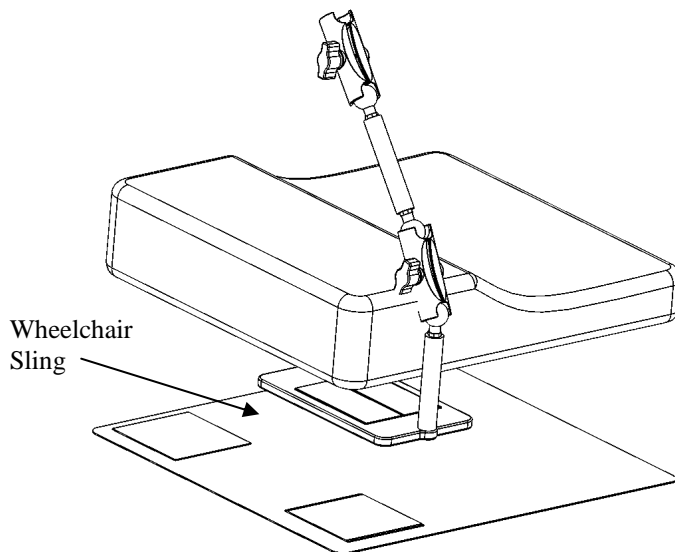

**Step 2:** Reinstall Cushion. Add device holder (sold separately) by loosening knob on upper clamp and install holder.

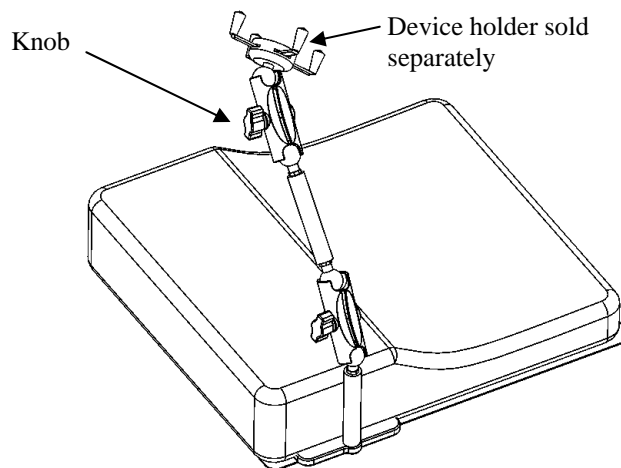


 Therafin Corporation  
9450 W Laraway Road  
Frankfort, IL 60423 USA  
800-843-7234  
[www.therafin.com](http://www.therafin.com)

## Seat Cushion Communication Mount


### To Adjust:

Loosen knobs and adjust to desired location.


Retighten knobs

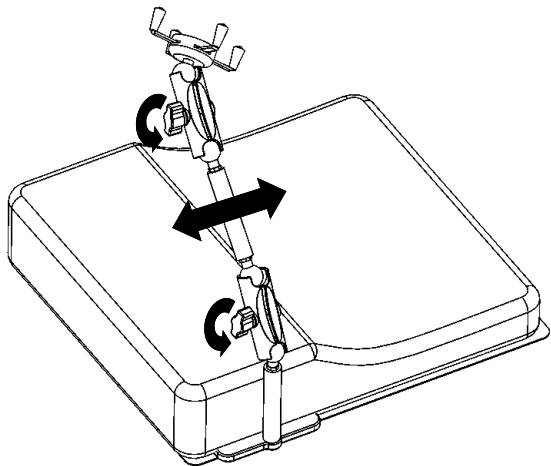

**Optional Configuration:** Remove Extension for a lower mounting alternative

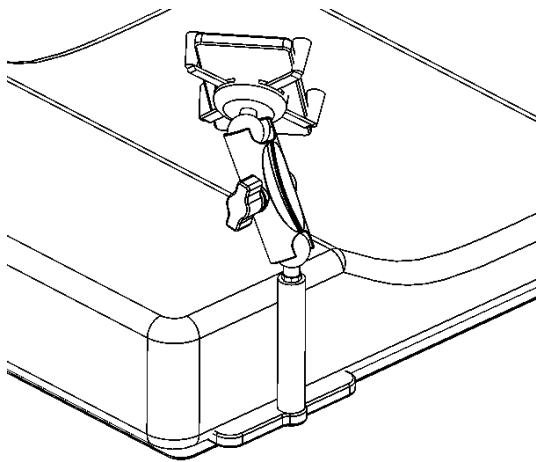

### Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.  
Air Dry.



2. Check all fasteners and retighten as needed.

### Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

*Document 08681  
Revision A 7-2016*



 Therafin Corporation  
9450 W Laraway Road  
Frankfort, IL 60423 USA  
800-843-7234  
[www.therafin.com](http://www.therafin.com)