



# SOCK STARTERS

## INSTRUCTIONS:


- 1) While seated, wedge top end of Sock Starter between knees (see illus.). Draw sock or stocking over unit — about half-way down plastic trough. **NOTE:** As sock is pulled on to soft flat Sock Starters — model nos. 30001 or 30010 — unit will form itself into a trough; rubber pad should be on outside arc of trough. Gather sock into notches to hold in place (illustrated at right).

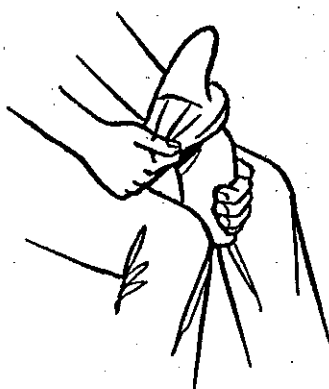

- 2) Attach garter hook to top of sock if needed (formed deluxe model only).
- 3) Position Sock Starter on floor in front of foot with toes toward opening.
- 4) Place foot onto plastic trough and insert toes through opening of sock or stocking.

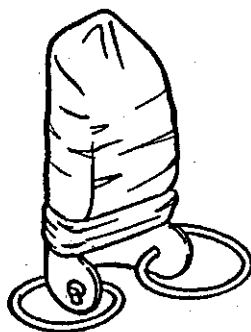

- 5) Pull back on rope or on plastic handle (formed deluxe model only) until toes are well into sock, manipulating foot and/or Sock Starter as necessary.

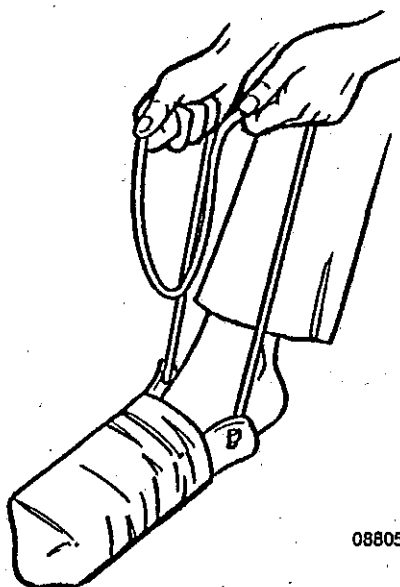

08805

- 6) Continue pulling under and around heel and slide unit up along calf allowing sock to slip off unit and onto foot.
- 7) Disconnect garter hooks from top of sock or stocking (formed deluxe model only).

 **Therafin**  
Corporation

9450 W Laraway Rd • Frankfort, IL 60423  
Customer 800-843-7234 • Fax 888-479-1515  
www.Therafin.com • info@Therafin.com