


VESTS

PARTS DESCRIPTION

- ◆ All parts shown here are not included with every vest
- ◆ Parts included vary depending on which vest style you have

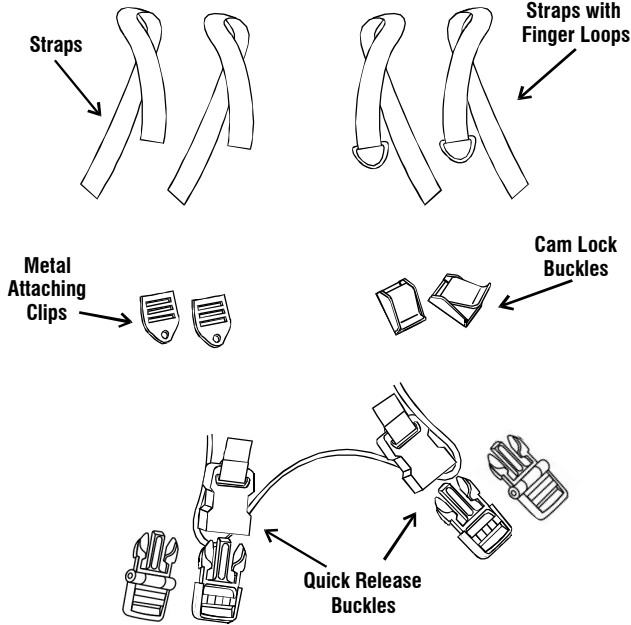


This vest is a **Kit** that you assemble as needed for your particular application. This sheet illustrates **examples** of the many ways to use the various straps, clips, and buckles.

HOW TO ATTACH A QUICK RELEASE BUCKLE TO A PLAIN STRAP

- ◆ Pull **Strap** for desired tight fit.



When threading a **Strap**, the **Buckle** piece must have the correct side up. The part the **Strap** goes around has a flat side that belongs on top, and a triangular side that belongs on the bottom.

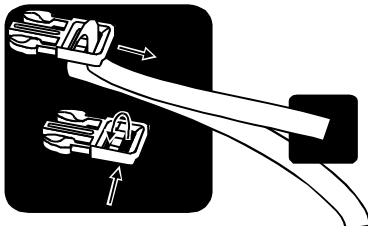


Thread the **Strap** as shown. Pull the top **Strap** to tighten, and it will stay tight in position against the **Buckle**.

HOW TO ATTACH A QUICK RELEASE BUCKLE TO A FINGER LOOP STRAP

- ◆ Use **Finger Loop** to pull **Strap** for desired tight fit.



When threading a **Strap**, the **Buckle** piece must have the correct side up. The part the **Strap** goes around has a flat side that belongs on top, and a triangular side that belongs on the bottom.

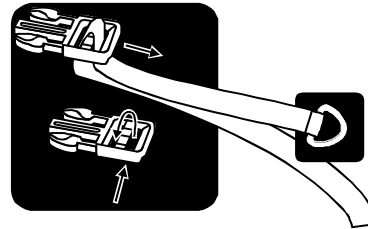


Thread the **Strap** so the **Finger Loop** is on the top. When you pull it, the **Strap** will stay tight in position against the **Buckle**.


HOW TO ATTACH A STRAP TO THE CHAIR

- ◆ Secure **Metal Attaching Clip** and/or **Cam Lock Buckle** to chair, then thread the strap as shown.

Metal Attaching Clips

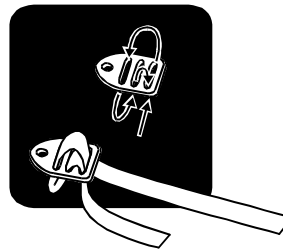

Cam Lock Buckles

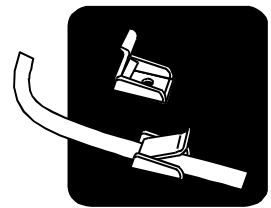

REMOVING USER FROM CHAIR

- ◆ Use **Quick Release Buckles** so you don't have to refit all the adjustments the next time user gets into chair.

08749

IMPORTANT NOTICE

- ◆ **THIS IS NOT A SAFETY DEVICE. IT IS TO BE USED ONLY FOR POSITIONING. IT MUST BE USED IN CONJUNCTION WITH A METAL SEAT BELT!**



9450 W Laraway Rd • Frankfort, IL 60423
 Customer 800-843-7234 • Fax 888-479-1515
 www.TheraFin.com • info@TheraFin.com