


FlexSure Feet Instructions

Medium Pair 31772
Large Pair 31773
X-Large Pair 31774

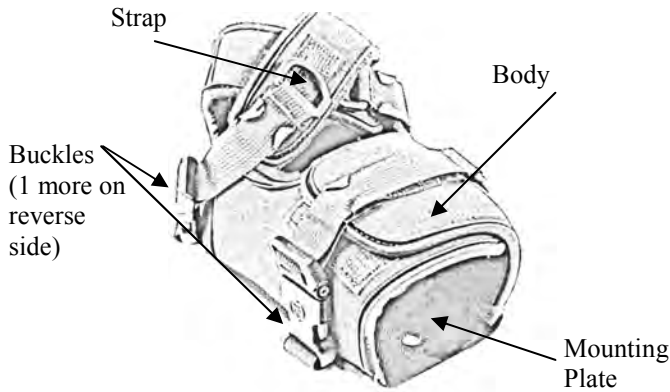

Install

Step 1 : Determine Location of the device on the users wheelchair footplate

Step 2 : Remove Mounting Plate and mark hole locations on the footplate. Drill all holes with 1/4" drill bit.




Step 3 : Place Mounting Plate back in Body and secure with included bolts and nuts.

08640

Operation

After inserting users foot, close front flap and secure buckle. Then cross the upper straps and secure with buckles. Adjust by pulling on the straps.

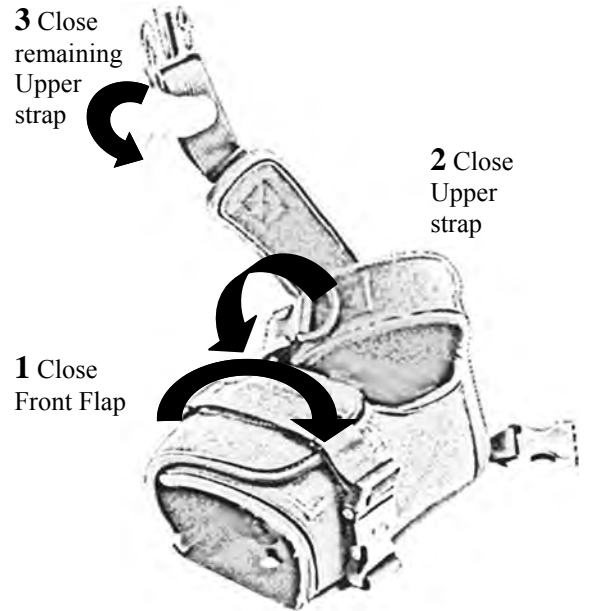

Over time, the hook surface can pick up lint and other debris that can affect its ability to grip the loop surface. Cleaning debris out of the hooks is similar to cleaning hairbrushes – and using a stiff brush or another piece of hook are the best methods. It can take some effort because the hooks in the tape are designed to grab. The optimal solution is prevention: the hook portion should be covered at all times, so it does not grab onto anything it should not – like a blanket or hair. Always engage the hook and loop before washing.



9450 W. Laraway Road • Frankfort, IL 60423
800 -843 -7234 • Fax: 888-479-1515
www.Therafin.com • info@Therafin.com