


Surelock Headrest Hardware

- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Product Identification

Components of the Surelock Headrest Device.

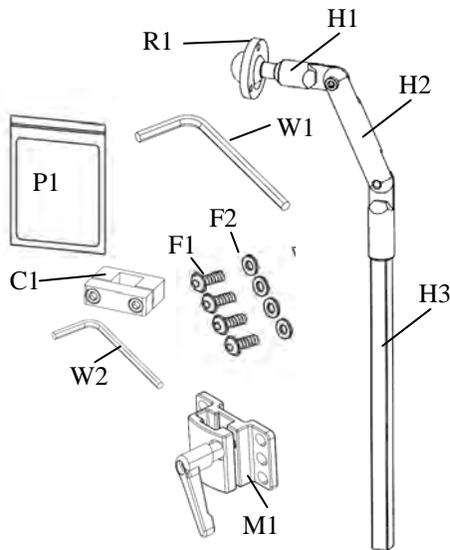


Clean before assembly and use


Components may have a protective coating to shield from environmental damage. Use wipe (P1) to clean components prior to assembly and use.

Installation and Assembly


1. Attach M1 to Wheelchair back using F1 and F2. Tighten securely with wrench W2.

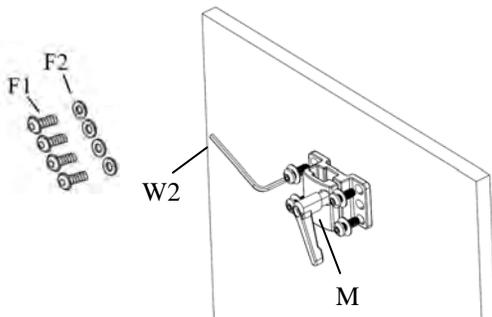


2. Place C1 onto H3 and insert H3 into M1. Tighten Knob.

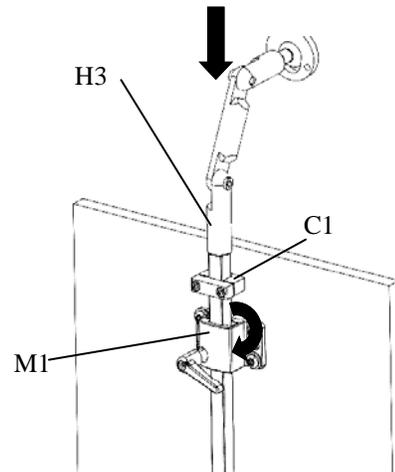

3. Place C1 in desired position and tighten bolts with wrench W2.

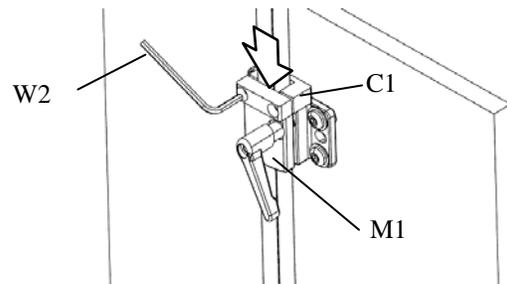

4. Attach headrest. (See Headrest Instructions)

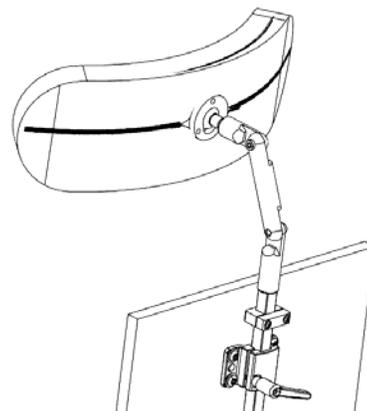


Surelock Headrest Hardware

Adjustment


- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Adjustments shown below. Re-tighten all fasteners after desired position is obtained.

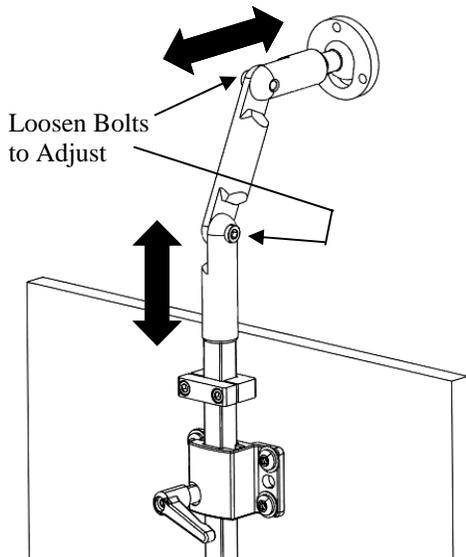

Device can be reconfigured by removing H2.

1. Remove bolts.
2. Remove H2 and attach H1 to H3.
3. Reinstall bolt and tighten.

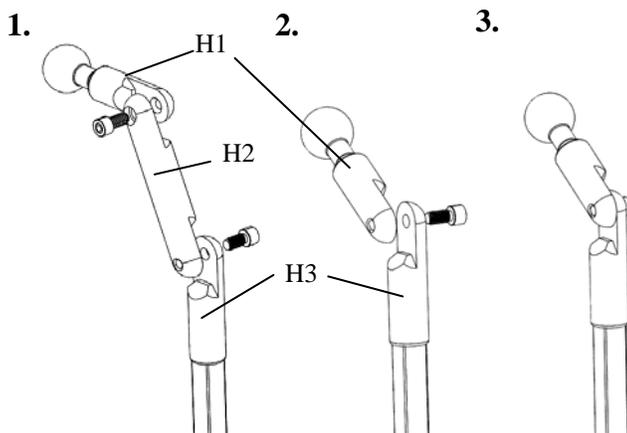

Care and Cleaning

- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.
Air Dry.



2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.



 Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60432 USA
800-843-7234
www.therafin.com