


ADJUSTABLE HEIGHT CHANNEL BRACKETS

Adjustable Height Flip-Away Model 31901A


Replacement Sliding Spacer, Pair 30301

Parts Description

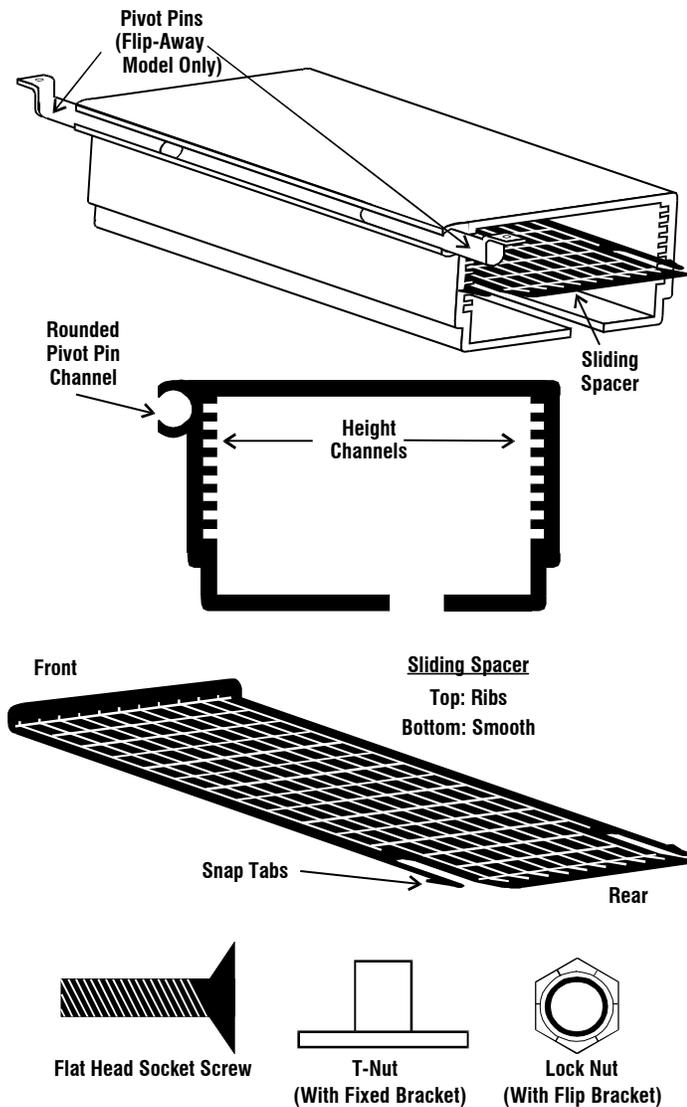


ADJUSTING BRACKET HEIGHT TO FIT ARMPAD THICKNESS



1. Slide bracket over armrest, making sure **Rounded Pivot Pin Channel** is on the OUTSIDE of the armrest.
2. Push the bracket up, so the bottom of the bracket touches the under-side of the armrest.

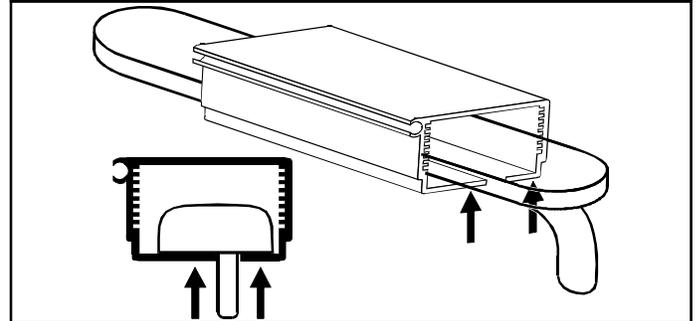

3. To insert the **Sliding Spacer** into the appropriate **Height Channel**, put the **Snap Tabs** in a channel, then push the **Sliding Spacer** completely into the bracket so that the tabs go completely through the bracket and snap into place. This allows the **Sliding Spacer** to be locked in place. **NOTE:** To choose the correct **Height Channel**, choose the channel that will allow the bracket to snugly fit on the armrest pad. The bracket must fit snugly on the armrest to provide a stable mounting foundation.

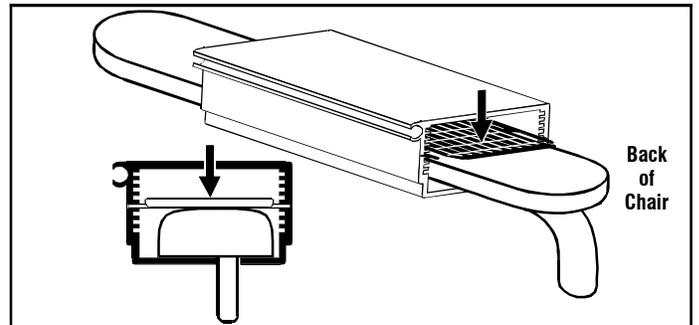


4. The **Snap Tabs** on either side of the **Sliding Spacer** **MUST BE INSTALLED INTO THE SAME SLOT ON EACH SIDE OF THE BRACKET.**

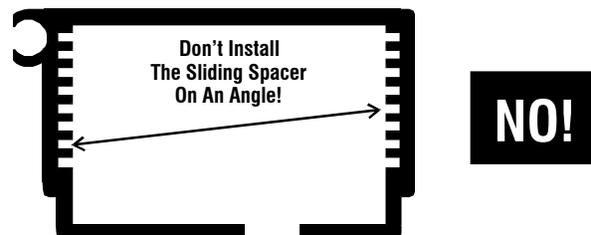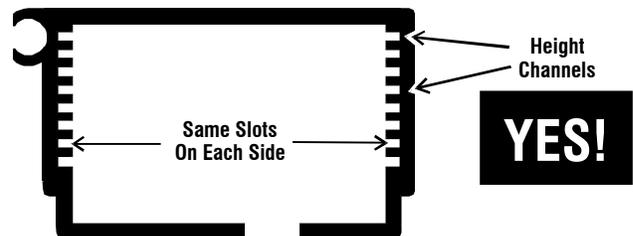

9450 W Laraway Rd • Frankfort, IL 60423
 Customer 800-843-7234 • Fax 888-479-1515
 www.Therafin.com • info@Therafin.com

You Tube Watch Instructional Video for this product at <http://www.youtube.com/TherafinCorporation>


5. To remove the **Sliding Spacer**, gently squeeze both **Snap Tabs** in, and push the Sliding Spacer back through the bracket.

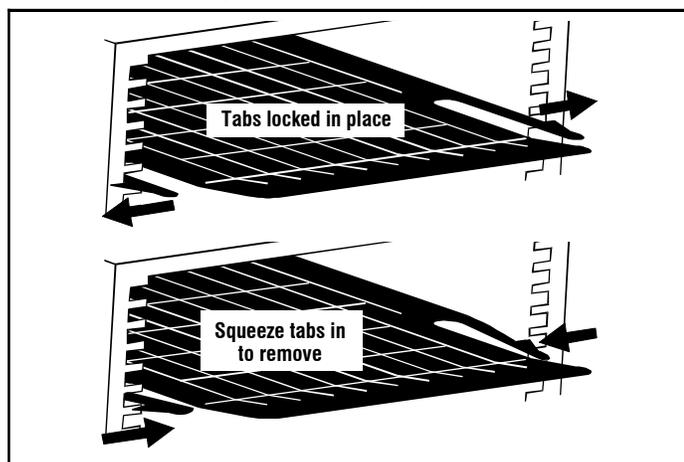

USING TWO SLIDING SPACERS

1. Two **Sliding Spacers** may be used together to provide maximum support in cases of extra weight or extreme pressure exerted on the half tray and bracket.
2. Adjust the bracket height to fit armpad thickness as shown on the reverse of this page. Clearly mark the slot you have chosen, so you don't change the snug fit on the armrest pad. Remove the **ORIGINAL Sliding Spacer**.
3. Install the **SUPPLEMENTAL Sliding Spacer** in the **Height Channel DIRECTLY ABOVE** (toward the tray) the slots you have marked.
4. Reinstall the **ORIGINAL Sliding Spacer** in the **Height Channel** you have marked.

NOTE: If the correct **Height Channel** to fit your armpad is the **VERY TOP SLOT**, you will not need to use two **Sliding Spacers**, as the bracket itself will provide the extra support needed.

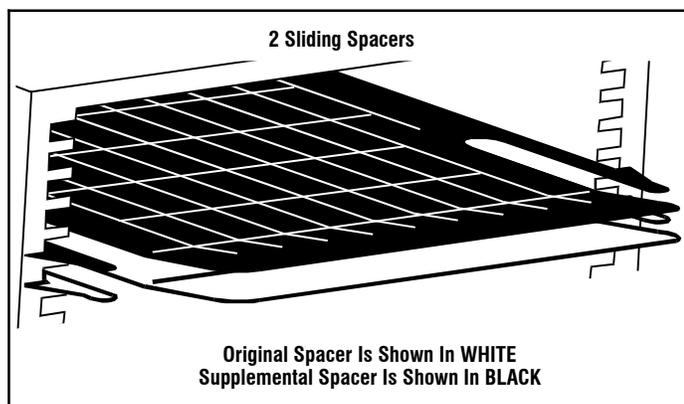


You Tube Watch Instructional Video for this product at <http://www.youtube.com/TherafinCorporation>


SECURING BRACKET TO TRAY

Flip-Away Bracket Installation:


1. Slide the bracket onto armrest, making sure **Rounded Pivot Pin Channel** is on OUTSIDE of armrest.
2. Slide one **Pivot Pin** into each end of the bracket channel, with the flat side facing up.

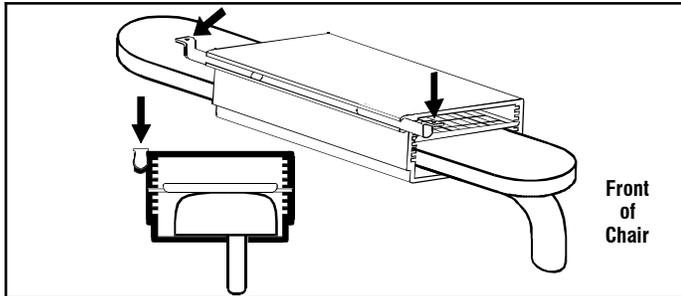


3. Properly position the tray on top of bracket, mark where the holes in **Pivot Pins** make contact with tray. **NOTE: If the tray is padded, you must determine the appropriate way to fasten the bracket to the underside of the tray, and you must supply the fastening hardware of your choice. If the tray is padded, do not follow step 4.**

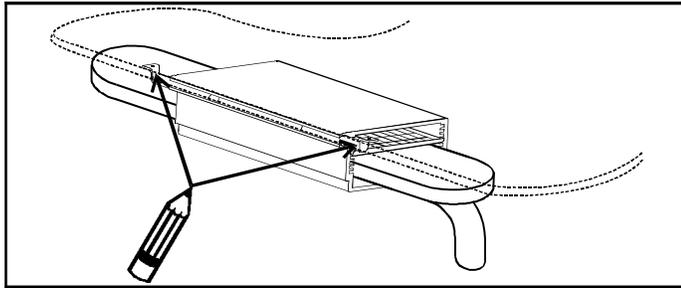

4. Remove tray and flip upside down on a flat working surface. Drill a 1/4" hole through the tray on each pivot pin hole mark. Countersink the holes on top side of tray, so that the socket head screw is flush with the surface of the tray. Remove bracket from armrest and reposition on tray, lining up holes in tray with holes in pivot pins.

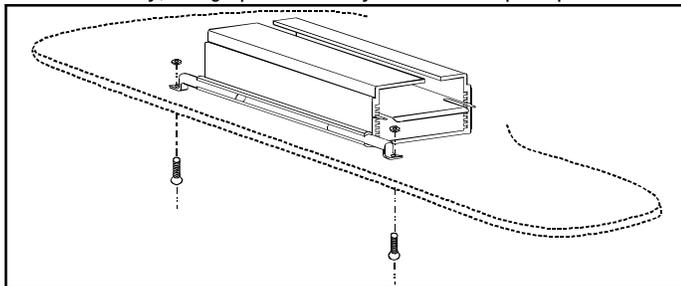

5. Attach the bracket to the tray using appropriate hardware.

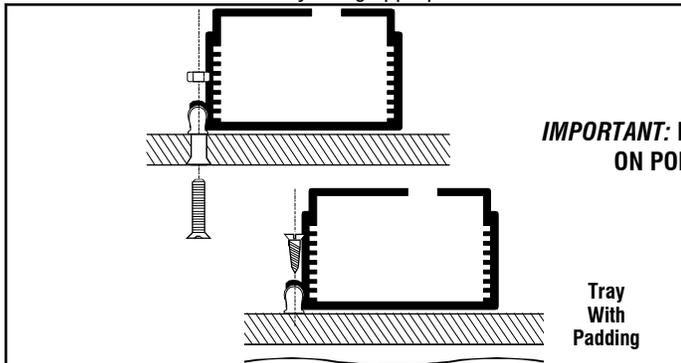

IMPORTANT: DO NOT USE LOCK TITE ON POLYCARB TRAYS!

6. Flip tray right-side up and slide onto armrest. The tray will now flip up and out of the way.

