

SNUG HUG

Full Body
Sensory Pressure Wrap



Be “Snug as a Bug in a Hug”

SMALL	30743	<input type="checkbox"/>	100% Spandex Nitrile Rubber Wrap
MEDIUM	30744	<input type="checkbox"/>	
LARGE	30745	<input type="checkbox"/>	
X-LARGE	30746	<input type="checkbox"/>	

SNUG HUG



Caution: Read instructions before operating device.



Caution: This device is not intended as a sleeping garment.



Caution: Exercise caution near open flames.



Caution: Do not wear device with straps located at the rear. Only wear as shown in the directions.

THE BLANKET SHOWN IN THESE INSTRUCTION PICTURES IS NOT INCLUDED IN THE "WRAP ONLY" VERSION YOU PURCHASED.



1. Unfold the wrap and set it out in the seating area. Separate the straps and lay them open.



2. Seat the user in the Snug Hug. Wrap the **Leg Hug** around the thighs.



3. Wrap the **Hip Hug** around the hips.



9450 W Laraway Rd
Frankfort, IL 60423
www.Therafin.com

800-843-7234
Fax 888-479-1515
info@Therafin.com



CARE INSTRUCTIONS

Snug Hug: Hand wash in cold water with mild detergent. Lay flat to dry.



4. Wrap the **Body Hug** on the torso.



5. Wrap the **Shoulder Hug** straps around each shoulder.

6. Re-tighten all straps as needed.

THE BLANKET SHOWN IN THESE INSTRUCTION PICTURES IS NOT INCLUDED IN THE "WRAP ONLY" VERSION YOU PURCHASED.

MORE INFORMATION

Adjusting to the user, **Folding** for on the go, and taking apart for **Washing**
<http://tinyurl.com/UsingSnugHug>



LIMITED WARRANTY. Therafin Corp. warrants that the Snug Hug product supplied by it shall be free from defects in material and workmanship for ninety (90) days from the date of delivery. This LIMITED WARRANTY does NOT apply where there has been a failure of the product due to improper use; breakage not due to defect or failure on account of faulty or improper use or handling. Said LIMITED WARRANTY IS EXCLUSIVE AND IS IN LIEU OF ANY OTHER REPRESENTATION OR WARRANTY, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This exclusion applies to original and replacement goods.

If you believe you have purchased defective goods as described above, you must notify Therafin Corp. at 9450 W Laraway Road, Frankfort, IL 60423 or phone 800-843-7234, immediately within the first ninety (90) days of ownership. Therafin may then request that you return the defective product to Therafin. If any goods are proven to be defective as described above, then Therafin will, at Therafin's sole discretion, either repair or replace the defective goods. Said repair or replacement of defective goods shall constitute fulfillment of all liabilities of Therafin to Buyer. Therafin reserves the right to change design, color, models, and to discontinue the manufacture of any goods.